

KARMAN HEALTHCARE INC.

19255 San Jose Ave., City of Industry, CA 91748

Tel: 1-800-805-2762 / 626-581-2235 Fax: 626 581-2335

www.karmanhealthcare.com Email: info001@karmanhealthcare.com



Part Catalog 2012



Table of Contents

PRODUCT	PAGE
Armrest Arm Pads, Anti-tippers, Batteries, & Cushions.....	3
Brake Tip.....	4
Canopy.....	5
Casters.....	6
Cushions.....	7
Elevating Leg Rest (ELR)	8
Head Rest.....	10
Heel Loop.....	11
Leg Strap.....	12
Push Bar.....	13
Push Bar, Hill Climber, Handgrip Foldable Head Rest.....	25
Seat Belt.....	26
Upholstery.....	2

Wheels.....28

The HIGH Quality Choice for the [Healthcare Industry](#).

We offer a wide variety of wheelchair replacement parts that covers the entire scope of our product line up that complements our OEM wheelchair selection. Karman proudly supplies more than 100 models and covers a full spectrum of mobility equipment as one of the leading wheelchair manufactures and provides support in parts replacement, accessories, and options that extend out of Karman's brand.

We have a comprehensive supply of parts for your upkeep, maintenance, and accessories for customization and allowing you peace of mind, safety, and enjoyment for your mobility equipment. We carry:

Wheelchair Upholstery, Positioning Belts, Tires and Tubes, Casters and Rear Wheels, Legrest and Footrests, Wheel Locks, Bed Pendants, and so much more

Thank you for your interest in our company.

1. Order Terms:

All prices are F.O.B. City of Industry. Terms are on sales order acknowledgements as well invoices.

2. Product Specifications and Prices:

We reserve the right to change product specifications and prices without prior notice. Karman Customer Service Department can provide pricing to authorized dealers only.

Prices are based on units as listed on the purchase order and/or invoice, with domestic commercial packaging.

3. Merchandise Inspection:

The customer upon delivery is required to carefully inspect all merchandise. Any visible damage or carton shortage should be noted on the carrier's delivery receipt. Claims for content shortage must be made to the Karman Customer Service Department within five (5) days of delivery or any such claim is waived. Concealed damage requires immediate inspection by the carrier or any damage claim is waived. The customer is responsible for the filing of all freight and concealed damage claims with the carrier.

4. Limited Warranty:

We warrant each item to be free of defects in material and workmanship under normal use and service for the length of our warranty as stated in our warranty policies. This warranty is in lieu of all other warranties expressed, implied or previously printed and does not apply in the event of neglect, abuse, misuse, or normal wear and tear. We are not responsible for injury that may result from abuse, misuse, or user negligence and our responsibility is limited to the original selling distributor. Unauthorized service modification of a product invalidates this warranty. If a product is deemed to be under warranty, Karman shall provide, at its option, one of the following: (1) replacement of any defective or nonconforming part or product or (2) a credit and/or refund of the original selling price made to Karman's initial customer on a prorated or depreciated basis. The warranty does not include any labor charges incurred in replacement part(s) installation or any associated freight or shipping charges to the manufacturer. Karman Customer Service for more information.

5. Shipments:

When placing an order, we will select the method of shipment for the customer. Any special shipping and packaging requests are the customer's responsibility

6. Freight Policy:

Freight charges apply on all orders, back orders or split shipments.

• **Inside delivery, lift gate, redelivers and special delivery circumstances**

are subject to additional charges.

• **Charges associated with redirecting shipments after an order has left the warehouse will be subject to additional freight charges.**

7. Return Goods Policy:

A Return Merchandise Authorization (RMA) number must accompany all returned merchandise. A Karman Customer Service Representative must authorize a return, issue a RMA number and provide a return shipping address in advance of return. Return authorizations are valid only for 30 days from the date of issue. Returned merchandise must be shipped prepaid freight (all collect shipments will be refused).

I. Eligibility:

- Request for returns due to customer error will only be accepted for 30 days after shipment date.
- Merchandise must be unused and in the original packaging as well as in new and resalable condition.
- Merchandise must be properly packaged, un-opened, in the same quantity as sold, and returned in the original shipping carton to avoid freight damage. Credit will not be given for items damaged due to improper packaging or freight damage.
- All returns are subject to inspection. We reserve the right to determine in our sole discretion whether an item qualifies for credit or for return as warranty, quality or defective goods (other than for freight damage) and whether to repair, replace or partially replace any such item.
- Refused Orders: Orders that are refused will be charged a 15% restock fee and all associated freight charges such as freight to the customer and back to a warehouse. This does not apply to orders refused due to freight damage.

II. Return Charges:

All returns, other than warranty and damaged items, are subject to a handling/restocking fee of 15% of the sale price of the item.

III. Non-Returnable Merchandise:

The following items may not be returned, other than those that are covered by warranty:

- Private label items
- Special orders (any item not stocked by Karman)

KARMAN HEALTHCARE INC.

19255 San Jose Ave., City of Industry, CA 91748

Tel: 1-800-805-2762 / 626-581-2235 Fax: 626 581-2335

www.karmanhealthcare.com Email: info001@karmanhealthcare.com



c. Custom or discontinued items

d. Any product that for hygienic reasons cannot be brought back into stock.

KARMAN HEALTHCARE INC.
 19255 San Jose Ave., City of Industry, CA 91748
 Tel: 1-800-805-2762 / 626-581-2235 Fax: 626 581-2335
www.karmanhealthcare.com Email: info001@karmanhealthcare.com



Anti-tippers
 Information – Short Version
 All are pairs.
 AT-800 – for KN-800B, LT-800B, LT-700B

Anti-Tippers



AT-950 looks like AT-115 but is shorter and has smaller hole. Note: smaller hole than AT-105.




AT-950 looks like AT-115 but is shorter and has smaller hole. Note: smaller hole than AT-105.

AT-802DY – for 802-DY



AT-802 – for KM802 Series (DISCONTINUED)
 Uses same as AT-105. Need a pen to push down the pin when in wheelchair.

AT-8020 – for KM8020 Series NOTICE: has no holes opposite side.

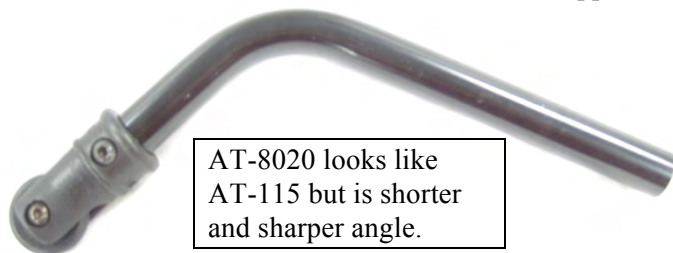

AT-8020 looks like AT-115 but is shorter and sharper angle.

AT-105 – for S-105



AT-105: Works with S-105, S-106, and KM-802. Note: S-115, S-105, S-106 & KM-802F all look like this now, only small differences in length, angle & thickness. Note2: S-106 comes with AT

AT-115 – for S-115



AT-115 looks like AT-950 but this is longer and has a wider hole. (DISCONTINUED)

AT-305 – for S-305



AT-980 - Anti-Tippers For Manual Wheelchairs
I T_080



Armrests

All armrests will fit most wheelchairs

Part #	Description
AP-105	Molded ureform pad, S105/115
AP-305	S-305
AP-950	LT950/980
AP21	Grey plastic full length armrests Size: 14 1/2" x 2 1/8"
AP22B	Unholstered, full arm pad, black Size: 13 3/4" x 2 1/4"
AP22U	Unholstered, full arm pad, dark blue Size: 13 3/4" x 2 1/4"
AP23B	Unholstered, desk arm pad, black Size: 10 1/4" x 2 1/8"
AP23U	Upholstered, desk arm pad, dark blue Size: 10 1/4" x 2 1/8"
AP88	Full armpad for KM-8000 (LT-980 too?)

Anti-tippers

All anti-tippers fit specific models.

Part #	Description	Model Fits
AT-105		S105,106,KM802,VIP-515
AT-115		S115
AT-305		S305
AT-800		KN800B,LT800B,LT700B
AT-802DY		KN802DY
AT-8020		KM8020
AT-980		LT980
AT-VIP		VIP-515
AT22	Standard bolt-on 7/8"	
AT800-DY		KN800DY
AT802		KM802
AT88		KM8020
AT9020L		KM9020L

Batteries

Part #	Description
BAT-12	12 volt BB cycle battery (12 Amp-12v Battery) Q – 4
BA-U1	12 volt UB12120 universal battery (U-1 12V 33AH Sealed Lead Acid) Q - 13

KARMAN HEALTHCARE INC.
 19255 San Jose Ave., City of Industry, CA 91748
 Tel: 1-800-805-2762 / 626-581-2235 Fax: 626 581-2335
www.karmanhealthcare.com Email: info001@karmanhealthcare.com



Part #	Description
CU-ERGO-16	Ergonomic memory 2" hgt. 16" x 16"
CU-ERGO-16-BACK	Ergonomic memory 16"
CU-ERGO-18	Ergonomic memory 2" hgt. 18" x 16"
CU-ERGO-18-BACK	Ergonomic memory 18"
CU-ERGO-20	Ergonomic memory 2" hgt. 20" x 16"
CU-ERGO-20-BACK	Ergonomic memory 20"
CU-GFR-16	Gel/foam combo 2 hgt. 16" x 16"
CU-GFR-18	Gel/foam combo 2 hgt. 18" x 16"
CU-GFR-20	Gel/foam combo 2 hgt. 20" x 16"
CU-FT-16	Foam combo 2 hgt. 16" x 16"
CU-FT-18	Foam combo 2 hgt. 18" x 16"
CU-FT-20	Foam combo 2 hgt. 20" x 16"
CU-XO202-BACK	Ergonomic Memory 18"
CU-XO202-SEAT	Ergonomic Memory 2" 18" x 16"

Cushions

KARMAN HEALTHCARE INC.
 19255 San Jose Ave., City of Industry, CA 91748
 Tel: 1-800-805-2762 / 626-581-2235 Fax: 626 581-2335
www.karmanhealthcare.com Email: info001@karmanhealthcare.com



Armrests



AP21



AP22B



AP22U



AP23B



AP23U

Anti-tippers



AT-115 (S-105, 115, 106, & KM-802F all look same)



AT-305



AT-800



AT-802DY



AT-8020



AT-980

Batteries



BAT-12



BA-U1

Cushions



Height Adjustable Armrests For MVP-502



AP21



Deluxe Universal Plastic Full Length Armrest. Universal hole spacing w/ 2 Screws.

Grey Color. Size: 14 1/2"(L) x 2 1/8"(W)

AP22B



Deluxe Universal Padded Full Length Arm Pads

Universal hole spacing w/ Screws

Black Color. Size: 13 3/4"(L) x 2 1/4"(W)

AP22U



Deluxe Universal Padded Full Length Arm Pads.

Universal hole spacing w/ 2 Screws

Blue Color. Size: 13 3/4"(L) x 2 1/4"(W)



Deluxe Universal Padded Desk Length Arm Pads.

Universal hole spacing w/ 2 Screws

Black Color. Size: 10 1/4"(L) x 2 1/8"(W)

KARMAN HEALTHCARE INC.
19255 San Jose Ave., City of Industry, CA 91748
Tel: 1-800-805-2762 / 626-581-2235 Fax: 626 581-2335
www.karmanhealthcare.com Email: info001@karmanhealthcare.com



AP23B



AP23U



Deluxe Universal Padded Desk Length Arm Pads.

Universal hole spacing w/ 2 Screws

Blue Color. Size: 10 1/4"(L) x 2 1/8"(W)

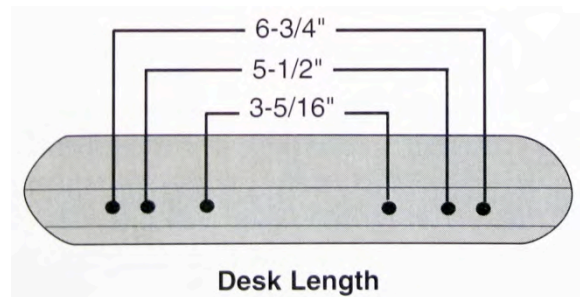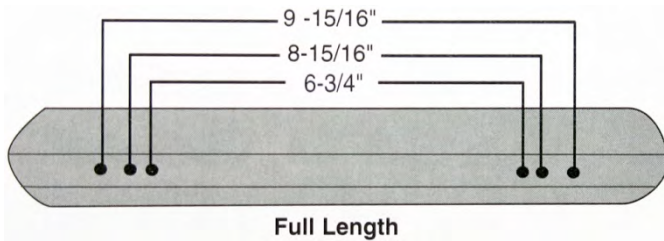

KARMAN HEALTHCARE Inc.

19255 San Jose Avenue . City of Industry . CA 91748-1418

Order.1-800-805-2762 Tel.626-581-2235 Fax.626-581-2335

KARMAN HEALTHCARE INC.
19255 San Jose Ave., City of Industry, CA 91748
Tel: 1-800-805-2762 / 626-581-2235 Fax: 626 581-2335
www.karmanhealthcare.com Email: info001@karmanhealthcare.com



Anti-tippers Information All are sold as pairs.

AT-950 – for LT-950/LT-980



AT-950

AT-802DY – for 802-DY (picture not shown, resembles to AT-802, same diameter, diff. height)

AT-802 – for KM802 Series AT-802 also for SOMA model.
7/8" diameter



AT-802

AT-8020 – for KM8020 Series.

AT-115-for S-115 AT-115 fits 7/8" frame. Also used for: S-105,106



AT-8020

AT-305 – for S-305

AT-305 fits 1" diameter frame, height adjustable.



AT-305



AT-115

Armrest Pads Information

AP21G – Plastic, full arm, grey, universal

AP22B – Upholstered, full arm pad, black universal

AP22U – Upholstered, full arm pad, blue universal

AP23B – Upholstered, desk arm pad, black universal

AP23U – Upholstered, desk arm pad, blue universal



AP21G

AP22B



AP23B

AP22U

AP23U

Canopy - Information

CANO-115 – Convertible Canopy for 7” / 8” Handles

Does not fit onto S-305



CANO-115

CB-2019

Carry Bag- Information

CB-2019 – Carry Bag for Transporter, approx. (L x W x H) 27” x 15” x 30”

All Gel / Form Cushion Information

CU-GFR-16 – gel / foam combo 16” x 16” 2” cushion

CU-GFR-18 - gel / foam combo 18” x 16” 2” cushion

CU-GFR-20 - gel / foam combo 20” x 16” 2” cushion

CU-FT-16 - foam 16” x 16” 2” cushion

CU-FT-18 - foam 18” x 16” 2” cushion

CU-FT-20 - foam 20” x 16” 2” cushion



CU-GFR-18



CU-FT-18

CU-ERGO-16 – ergonomic memory 2” height 16” x 16” seat cushion

CU-ERGO-18 - ergonomic memory 2” height 18” x 16” seat cushion

CU-ERGO-20 - ergonomic memory 2” height 20” x 16” seat cushion

CU-ERGO-16-BACK - ergonomic memory 16” back cushion

CU-ERGO-18-BACK - ergonomic memory 18” back cushion

CU-ERGO-20-BACK - ergonomic memory 20” back cushion



CU-ERGO-18-BACK & SEAT
(same for XO-202)

CU-XO202-SEAT – memory seat 2” height 18” x 17” ergonomic cushion for stand-up wheelchair

CU-XO202-BACK - memory back ergonomic cushion for stand-up wheelchair

Elevating Legrest for Ergonomics and Recliner/Tilt Information

EL-S115-18 – for S-115 18” width

EL-S115-16 – for S-115 16” width

EL-S305-18 – for S-305 18” width

EL-S305-16 – for S-305 16” width

EL-VIP515-18 – for VIP515 18” Width (Also compatible w/ MVP 502)

EL-VIP515-16 – for VIP515 16” Width (Also compatible w/MVP 502)

Note: Elevating leg rest models listed above are in Diamond Black Color ONLY.

EL-VIP515 (shown)



Headrest / Backrest Extension Information

HR-FLD-115 – Foldable Headrest for 7” / 8” Handle Frame

BKR-EXT-S105-16 – Backrest extension S-105/115 16”

BKR-EXT-S105-18 – Backrest extension S-105/115 18”

BKR-EXT-S305-16 – Backrest extension S-305 16”

BKR-EXT-S305-18 – Backrest extension S-305 18”



Push Handles / Push Bars Information –

HD-XO202– push handles for stand-up XO-101 / XO-202

PB-115-16– foldable rigidifying push bars for 16” width

PB-115-18– foldable rigidifying push bars for 18” width

Note: Available ONLY for VIP, S-105, 106, 115



Seat Belts Information –

SB22 – Two-Piece Seat Belt. 45 “ Long, 2” Webbing.

SB88 – One-Piece Seat Belt. 60” Long, 2” Webbing.

SB99 – Seat Belt for Power Wheelchair (Push Button, auto style.60” long, 2” webbing)

SB99-48 Seat Belt for Manual Wheelchair (Push Button, Auto style 48” long, 2” webbing)




Caster Suspension Information –

FROG LEGS Suspension/FRO- (available for S-105,115,106,305)

Absorbs 76% of Jolts & Vibrations. Lighter and 10 x stronger than traditional fork. Comes with 6” x 1 1/4” Casters.

Hill Climber Attachment Information –

HCB- Our patented brake design attaches below original brake
 (Extension to S-305 only) allows for rollback-free hill climbing.

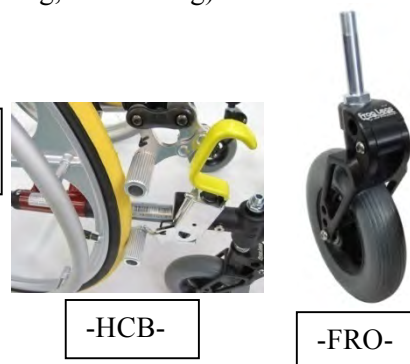

Upholstery – Ergonomics & VIP Information –



UB-S115 – removable upper back for 105 & 115 18” AEGIS Coating

UB-S115N - removable upper back for 105 & 115 16” AEGIS Coating

UB-S115W - removable upper back for 115W 20” AEGIS Coating

UB-S305 - removable upper back for 305 18” AEGIS Coating

UB-S305N - removable upper back for 305N 16” AEGIS Coating

UB-VIP515 - removable upper back for VIP-515 18”

UB-VIP505N - removable upper back for VIP-515 16”

US-S115 - removable upper seat for 105 & 115 18” AEGIS Coating

US-S115N - removable upper seat for 105 & 115 16” AEGIS Coating

US-S115W - removable upper seat for 115W 20” AEGIS Coating

US-S305 - removable upper seat for 305 18” AEGIS Coating

US-S305N - removable upper seat for 305N 16” AEGIS Coating

US-VIP515 - removable upper seat for VIP-515 18” AEGIS Coating

US-VIP515N - removable upper seat for VIP-515 16” AEGIS Coating

UB-S305 & US-S305



UB-VIP515, US-VIP515 & VIP515 Head Pillow

Wheels for Ergonomic Information –

RF-115 – reflection wheel cover for s-ergo 105, 106, 115, 305 (Non-Ergo hand rim)

RW-MGN-SY – magnesium wheels, 24” x 1” yellow or grey tires Q/R axles, Silver rim (S-305 & S-115Q only)

RW-VLCN-SG – vulcan wheels, 24” x 1” grey tires, Q/R axles, Silver Rim. (S-305 & S-115 only)



RF-115



RW-VLCN-SG



RW-MGN-SY

Brake Tip Extension For Ergonomic Manual Wheelchairs



KARMAN HEALTHCARE INC.
19255 San Jose Ave., City of Industry, CA 91748
Tel: 1-800-805-2762 / 626-581-2235 Fax: 626 581-2335
www.karmanhealthcare.com Email: info001@karmanhealthcare.com



BT305 – BRAKE TIP FOR S-ERGO305Q



KARMAN HEALTHCARE Inc.

19255 San Jose Avenue . City of Industry . CA 91748-1418
Order. 1-800-805-2762 Tel. 626-581-2235 Fax. 626-581-2335




CANO-115

Multi-Function Deluxe Canopy

- Canopy can rotate from back to front over 180°
- Canopy constructed with UV resistant materials
- Allows compact folding while still attached to the wheelchair
- Can fit from 16"- 20" back width & only 7/8" diameter frame
- Lightweight frame & with length-wise folding function, can be folded behind the backrest.
- Height & Angle Adjustable



Note: Does not fit onto S-305

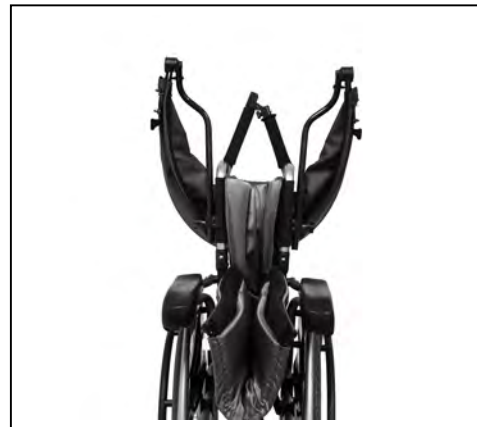

Frog Legs™ Shock Absorbers Installation Instructions

Materials Required for Installation

- 1 pair of Frog Legs suspension caster fork
- 1 installation hardware pack (supplied with forks)

Tools Required To Install Frog Legs™ Suspension Caster Fork

- Small flat head (regular) screwdriver
- 3/4 inch (19mm) socket and ratchet

Tools Required To Install Wheel or Suspension Dampener

- 2- 5/32 inch (4mm) Allen wrenches

Directions for Installing Frog Legs Shock Absorbers

Step 1. Use a small screwdriver to remove the plastic cover seam between the cap and caster barrel, and pry up with the screw driver. (See diagram above for caster fork details.)

Step 2. Use a 3/4 inch (19mm) socket and ratchet to remove the rigid fork assembly. The rigid fork assembly will have to be held in place to remove the nut. Usually this can be held by hand work. Once the nut is removed, the rigid fork assembly can be removed.

Note: The "nylock" nut, washers, and stem bearings will be used to install your new suspension forks. New nylock nuts and washers are supplied with your order if needed.

Step 3. If your rigid fork caster wheel is 6 inches in diameter or less and you would like to use this wheel for your new suspension system, then this wheel has to be removed. Depending on the type of rigid system you currently have, you may have to find some additional tools to remove the caster wheel. Once the wheel is removed, it can be installed on your new Frog Legs Shock Absorbers.

Supplied with your new set are (2) axle pins, (4) button-head screws and (4) wheel spacers you will need to attach your new caster wheels to your new forks. First choose one of the three axle location holes in your new forks, and then put the axle bolt partially through the hole. Add one of the provided spacers, or use your rigid system spacer if you have them. Next, insert the wheel, push the axle through the wheel and then add another spacer. Push the axle bolt through the other side of the fork. Install the "button-head screw" that came with the axle pin and tighten the assembly.

Step 4. Install the suspension fork assembly on the chair by inserting the stem of the suspension assembly into the caster barrel housing on your chair. See above diagram. Install the new "nylock" nut on the stem bolt with the 3/4 inch (19mm) socket and ratchet. Tighten this nut until all vertical stem play is removed. Over tightening will result in a fork assembly that will not swivel freely. If this occurs, simply loosen the stem "nylock" nut until fork assembly rotates freely.



Frog Legs™ Shock Absorbers Installation Instructions

Materials Required for Installation

- 1 pair of Frog Legs suspension caster fork
- 1 installation hardware pack (supplied with forks)

Tools Required To Install Frog Legs™ Suspension Caster Fork

- Small flat head (regular) screwdriver
- 3/4 inch (19mm) socket and ratchet

Tools Required To Install Wheel or Suspension Dampener

- 2- 5/32 inch (4mm) Allen wrenches

Directions for Installing Frog Legs Shock Absorbers

Step 1. Use a small screwdriver to remove the plastic cover or dust cap from the top of the caster barrel housing. Find the seam between the cap and caster barrel, and pry up with the screw driver. (See diagram above for caster barrel details.)

Step 2. Use a 3/4 inch (19mm) socket and ratchet to remove the "nylock" nut located below the dust cap removed in Step 1. The rigid fork assembly will have to be held in place to remove the nut. Usually this can be held by hand, but bracing the assembly against a hard surface will also work. Once the nut is removed, the rigid fork assembly can be removed from the caster barrel housing.

Note: The "nylock" nut, washers, and stem bearings will be used to install your new suspension forks. New nylock nuts and washers are supplied with your order if needed.

Step 3. If your rigid fork caster wheel is 6 inches in diameter or less and you would like to use this wheel for your new suspension system, then this wheel has to be removed. Depending on the type of rigid system you currently have, you may have to find some additional tools to remove the caster wheel. Once the wheel is removed, it can be installed on your new Frog Legs Shock Absorbers.

Supplied with your new set are (2) axle pins, (4) button-head screws and (4) wheel spacers you will need to attach your new caster wheels to your new forks. First choose one of the three axle location holes in your new forks, and then put the axle bolt partially through the hole. Add one of the provided spacers, or use your rigid system spacer if you have them. Next, insert the wheel, push the axle through the wheel and then add another spacer. Push the axle bolt through the other side of the fork. Install the "button-head screw" that came with the axle pin and tighten the assembly.

Step 4. Install the suspension fork assembly on the chair by inserting the stem of the suspension assembly into the caster barrel housing on your chair. See above diagram. Install the new "nylock" nut on the stem bolt with the 3/4 inch (19mm) socket and ratchet. Tighten this nut until all vertical stem play is removed. Over tightening will result in a fork assembly that will not swivel freely. If this occurs, simply loosen the stem "nylock" nut until fork assembly rotates freely.

PSG35F

Manual Wheelchair *Shock Absorbers*



- Great for active users wheelchair!
- User benefits from having shock absorbers installed already when they receive they're chair.
- Shock absorbing fork, absorbs 76% of jolts & vibration that affect the ride of users wheelchair
- Replaces the rigid caster forks that come standard with wheelchair.
- Lighter & 10 times stronger than traditional caster forks.
- Lowers floor-to-seat height to 18"
- Comes with 6"x1" solid flat free polyurethane front casters
- Available in black aluminum
- Lifetime limited warranty
- Supports weight ranges from: 120-160 lbs (PSG35F60) & 160-220 lbs (PSG35F61)



KARMAN HEALTHCARE INC.

19255 San Jose Ave., City of Industry, CA 91748

Tel: 1-800-805-2762 / 626-581-2235 Fax: 626 581-2335

www.karmanhealthcare.com Email: info001@karmanhealthcare.com



Excellence Through Mobility



5" A

5" F

6" C46

6" C46flat

6" C6141LT

7" C71LT



8" C81G

8" C81B-BT

8" B

8" C-20050

8" C

8" C81B



8" G

8" x 2" H

12"

12"

Rear Wheels



RW-MGN-SY

RW-VLCN-SG

Low-Profile Wheel

High-Profile Wheel

All casters are available in either 5/16" or 7/16" bearing size, ask if in stock

Part #	Description	Tread Type
A	Grey 5" x 1" tire w/ black 5-spoke hub	Smooth
C46	Grey 6" x 1" tire w/ black 7-spoke hub	Smooth
C46flat	Grey 6" x 1" tire w/ black 7-spoke hub	Round Flat
B	Black 8" x 1 1/2" tire w/ black 3-spoke hub	Rib
C-20050/WL-3600	Grey 8" x 1 1/2" tire w/ black 6-spoke solid hub	Rib
C	Grey 8" x 1 1/2" tire w/ black 5-spoke hub	4-Center Rib
D	Black 12" x 2" tire w/ black 5-spoke hub	Rib
E	Grey 12" x 2" tire w/ black 5-spoke hub	Rib
C81G	Black 8" x 1" tire w/ black 5-spoke hub, for Invacare	Smooth
C81B-BT	Grey 8" x 1" tire w/ black 5-spoke hub, for Invacare	Smooth
C81B	Grey 8" x 1" tire w/ black 10-spoke hub	Smooth
C614LT	Grey 6" x 1" tire w/ black 3-spoke hub, for Karma	Smooth
C71LT	Grey 7" x 1" tire w/ black 6-spoke hub, for Karma	Smooth
C6141LT	Grey 6" x 1" tire w/ black 3-spoke hub, for KM-9020L	Rib
F	Grey 5" x 1" tire w/ black 3-spoke hub, for Karma	Smooth
G	Grey 8" x 1" tire w/ black 5-spoke hub, for Karma	Smooth
C61LT	Grey 6" x 1" tire w/ black 3-spoke hub	Smooth
C81-HD		
C81B-H6		
C81B-R		
C81L1		
C82B		
C82LT		
CASTER-4600		
CASTER-5		
CASTER-6		
CASTER-7		
CASTER-8-PNEU		
C71LT3	Grey 7" x 1" tire w/ black 3-spoke hub	Smooth, bumps
C71LT6	Grey 7" x 1" tire w/ black 7-spoke hub	Smooth
C81B8	Grey 8" x 1" tire w/ black 8-spoke hub	Smooth
C81B10	Grey 8" x 1" tire w/ black 10-spoke hub	Smooth
RW12PU		
CASTER2	Small air-fill 5 circle holes, no spokes	
H	Grey 8" x 2" tire w/ black 6-spoke solid hub	Bariatric 8"
I	Grey 6" x 1" tire w/ black 5-spoke hub Rib	XO-101 front
PSG35F61-BLK	Blk shock absorbers, 3.5" stembolt wt. 160-220lbs	S305 q-1
C51LT	5" x 1" lt wt caster polyurethane	q-9
C141LT	6" x 1.25" lt wt caster PU (actually C61141LT)	q-169
PSG25F60-BLK	Black shock absorbers, 2.5" stembolt wt. 120-160 lbs.	S105/115 q-1
PSG25F61-BLK	Black shock absorbers, 2.5" stembolt wt. 160-220 lbs.	S105/115 q-2
PSG35F60-BLK	Black shock absorbers, 3.5" stembolt wt. 120-160 lbs.	S305 q-1
PSG35F61-BLK	Black shock absorbers, 3.5" stembolt wt. 160-220 lbs.	S305 q-1

KARMAN HEALTHCARE INC.
19255 San Jose Ave., City of Industry, CA 91748
Tel: 1-800-805-2762 / 626-581-2235 Fax: 626 581-2335
www.karmanhealthcare.com Email: info001@karmanhealthcare.com



All Gel / Form Cushion

CU-GFR-16 – gel / foam combo 16” x 16” 2” cushion

CU-GFR-18 - gel / foam combo 18” x 16” 2” cushion

CU-GFR-20 - gel / foam combo 20” x 16” 2” cushion

CU-FT-16 - foam 16” x 16” 2” cushion

CU-FT-18 - foam 18” x 16” 2” cushion

CU-FT-20 - foam 20” x 16” 2” cushion



CU-ERGO-16 – ergonomic memory 2” height 16” x 16” seat cushion

CU-ERGO-18 - ergonomic memory 2” height 18” x 16” seat cushion

CU-ERGO-20 - ergonomic memory 2” height 20” x 16” seat cushion

CU-ERGO-16-BACK - ergonomic memory 16” back cushion

CU-ERGO-18-BACK - ergonomic memory 18” back cushion

CU-ERGO-20-BACK - ergonomic memory 20” back cushion



KARMAN HEALTHCARE INC.
19255 San Jose Ave., City of Industry, CA 91748
Tel: 1-800-805-2762 / 626-581-2235 Fax: 626 581-2335
www.karmanhealthcare.com Email: info001@karmanhealthcare.com



Ergo Fit Cushion

Ergonomic Memory Cushion



Ergonomic Memory Seat Cushion

16" () CU1616-SEAT

18" () CU1817-SEAT

20" () CU2017-SEAT

Official Products Compatible with KARMA Ergonomic


Wheelchairs: S-Ergo105, S-115,S-125,S-305, S-Ergo106, Ergo Flight, Ergo Lite, All ERGO.

Ergonomic Memory Back Cushion

16" () CU16-BACK

18" () CU18-BACK

20" () CU20-BACK

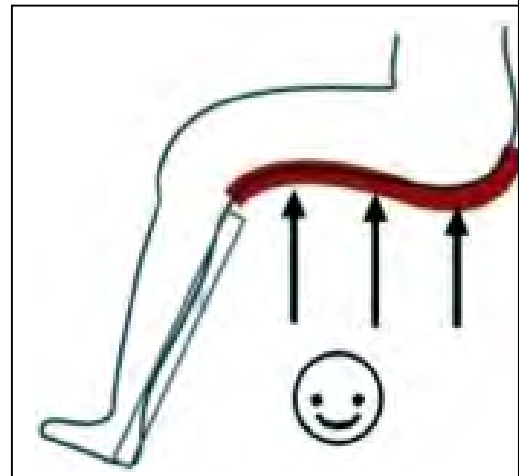

KARMAN HEALTHCARE INC.

19255 San Jose Ave., City of Industry, CA 91748

Tel: 1-800-805-2762 / 626-581-2235 Fax: 626 581-2335

www.karmanhealthcare.com Email: info001@karmanhealthcare.com



Karman Healthcare Inc.

19255 San Jose Ave.

City of Industry, CA 91748

Tel: 1-800-805-2762

www.karmanhealthcare.com

CU-Ergo

Exclusive Karman Contoured Memory Foam Cushion

- **Contoured Memory Foam cushion**
- **Non-slip bottom**
- **Hand washable**
- **2" Memory foam**
- **Simple to clean, inside and out**
- **Available in 16x17 or 18x17 or 20x17**
- **Color: Black**
- **Fire Retardant**
- **Distributed by the providers of the Ergonomic Wheelchair Line**



KARMAN HEALTHCARE

Exclusive Karman Wheelchair Gel/Foam Cushion

- **Foam cushion w/ gel insert**
- **Incontinent top**
- **Non-slip bottom**
- **Hand washable**
- **2" thick foam**
- **Simple to clean, inside and out**
- **Available in 16x16 or 18x16**
- **Comes in black top & off-white bottom**
- **Distributed by the providers of the Ergonomic Wheelchair Line**
- **Made in USA**



KARMAN HEALTHCARE Inc.

19255 San Jose Avenue . City of Industry . CA 91748-1418
Order.1-800-805-2762 Tel.626-581-2235 Fax.626-581-2335

KARMAN HEALTHCARE INC.
19255 San Jose Ave., City of Industry, CA 91748
Tel: 1-800-805-2762 / 626-581-2235 Fax: 626 581-2335
www.karmanhealthcare.com Email: info001@karmanhealthcare.com



Elevating Legrests for Ergonomic Wheelchairs

Fits: S-Ergo115 Ergonomic Manual Wheelchairs



Footrests



FR18B (E&J)

FR18C-EJ-DY (E&J)

FR18B-INV-DY



FR-S115R-18



FR-S305R-18



FR-VIP515-18

Elevated Footrests



EL18VR??



EL18BB-EJ-DY



EL-18CB-INV-DY??



EL-S305R-18 (EL-VIP515 looks same)



EL-S115-18

Footrests

All footrests fit specific types of models.

Part #	Description	Model Fits
FR-9020L	FR:KM-9020L	q- 0
FR-S115R-16	FR: S-115 16" Red	q- 0
FR-S115R-18	FR: S-115 18" Red	Q- 0
FR-S115S-16	FR: S-115 16" Silver	Q- 0
FR-S115S-18	FR: S-115 18" Silver	Q- 1
FR-305R-16	FR: S-305 16" Red	Q- 0
FR-305R-18	FR: S-305 18" Red	Q - -3
FR-S305S-16	FR: S-305 16" Silver	Q - 0
FR-S305S-18	FR: S-305 18" Silver	Q - -1
FR-VIP515-16	FR: VIP 16"	Q- 0
FR-VIP515-18	FR: VIP 18"	Q- 0
FR16	FR: 16"	Q- 3
FR16C-EJ-DY	E&J chrome 16"	Q- 14
FR16HL	FR: 16" hemi-left	Q- 9
FR18	FR: 18"	Q- -1
FR18B	E&J black	Q- 9
FR18B-INV-DY	FR: Gap: 1 3/8" Black	Q- 26
FR18C-EJ-DY	E&J Chrome 18"	Q- 5
FR8000-BK	FR: Alum. Black KM8000	Q- -2

Elevated Footrests

Part #	Description	Model Fits
EL-S115-18	ELR: S-115 18" Fits KM802	
EL-S305R-16	ELR: S-305 Red 16"	
EL-S305R-18	ELR: S-305 Red 18"	
EL-S305S-16	ELR: S-305 Silver 16"	
EL-S305S-18	ELR: S-305 Silver 18"	
EL-VIP515	ELR: VIP515 18"	
EL-VIP515-16	ELR: VIP515 16"	
EL16	ELR: Padded 16"	Q- -1
EL16BB-INV-DY	ELR: 16" Black hemi-INV	Q - 34
EL16VR	ELR: 16" c/p Black pad alu footpt	Q - 3
EL18	ELR: c/p, nlk pad, 18" E&J	Q- 14
EL18BB-EJ-DY	ELR: E&J 3 1/8" Gap	Q - 60
EL18BB-INV-DY	ELR: footrest blk frame/pad hemi-INV	Q- 64
EL18CB-EJ-DY	ELR: c/p black pad E&J	Q - 10
EL18CB-INV-DY	ELR: c/p blk pad hemi-INV	Q- 31
EL18CU-INV-DY	ELR: c/p blue pad hemi-inv	Q - 57
EL18VR	ELR: c/p blk pad INV	Q- 9
EL8000	ELR: padded KM8000	Q- 8

KARMAN HEALTHCARE INC.

19255 San Jose Ave., City of Industry, CA 91748

Tel: 1-800-805-2762 / 626-581-2235 Fax: 626 581-2335

www.karmanhealthcare.com Email: info001@karmanhealthcare.com



EL8000UL	ELR: alum blue left (U is 0)	Q- 8 (R&L are 8 each)
----------	------------------------------	-----------------------

KARMAN HEALTHCARE INC.
19255 San Jose Ave., City of Industry, CA 91748
Tel: 1-800-805-2762 / 626-581-2235 Fax: 626 581-2335
www.karmanhealthcare.com Email: info001@karmanhealthcare.com



Elevating Legrest for S-Ergo and Tilt

Information

EL-S115-18 – for s-115 18” width, black.

EL-S305R-18 – for s-305 18” width, red.

EL-S305R-16 – for s-305 16” width, red

EL-S305S-18 - for s-305 18” width, silver

EL-S305S-16 - for s-305 16” width, silver

EL-VIP515-18 – for VIP515 18” Width

EL-VIP515-16 – for VIP515 16” Width

KARMAN HEALTHCARE INC.
19255 San Jose Ave., City of Industry, CA 91748
Tel: 1-800-805-2762 / 626-581-2235 Fax: 626 581-2335
www.karmanhealthcare.com Email: info001@karmanhealthcare.com



Elevating Legrests for Manual Wheelchair



Elevating Legrests for Ergonomic Wheelchair

- Composite footrests & padded calf pad are also included in your purchase of elevating legrest!
- Height adjustable footplates
- Articulating hinges allow for less stress on joints when elevating legs.
- Anti-microbial paint allows for easy clean everytime.
- Composite footrests & padded calf pad are also included in your purchase of elevating legrest!





HR-FLD-115

Multi-Function Deluxe Headrest

- Headrest component can rotate 360°, up/down & back or forth.
- Allows compact folding while still attached to the wheelchair
- Can fit from 16"- 20" back width on 7/8" diameter frame
- Lightweight frame & sturdy padding
- Easy to maintain
- Reaches heights from seat 18" to 23"

How to use

When using wheelchair, unfold wheelchair first, then prop up the headrest. Conversely when folding the wheelchair, first pull down the headrest to release the support tension, then fold wheelchair.



KARMAN HEALTHCARE INC.
19255 San Jose Ave., City of Industry, CA 91748
Tel: 1-800-805-2762 / 626-581-2235 Fax: 626 581-2335
www.karmanhealthcare.com Email: info001@karmanhealthcare.com



KARMAN HEALTHCARE Inc.

19255 San Jose Avenue . City of Industry . CA 91748-1418
Order.1-800-805-2762 Tel.626-581-2235 Fax.626-581-2335

KARMAN HEALTHCARE INC.
19255 San Jose Ave., City of Industry, CA 91748
Tel: 1-800-805-2762 / 626-581-2235 Fax: 626 581-2335
www.karmanhealthcare.com Email: info001@karmanhealthcare.com



HL18/HL16/HL20 –HEEL LOOP FOR MANUAL WHEELCHAIR




KARMAN HEALTHCARE Inc.

19255 San Jose Avenue . City of Industry . CA 91748-1418
Order.1-800-805-2762 Tel.626-581-2235 Fax.626-581-2335

KARMAN HEALTHCARE INC.
19255 San Jose Ave., City of Industry, CA 91748
Tel: 1-800-805-2762 / 626-581-2235 Fax: 626 581-2335
www.karmanhealthcare.com Email: info001@karmanhealthcare.com



Leg Straps For Manual Wheelchairs

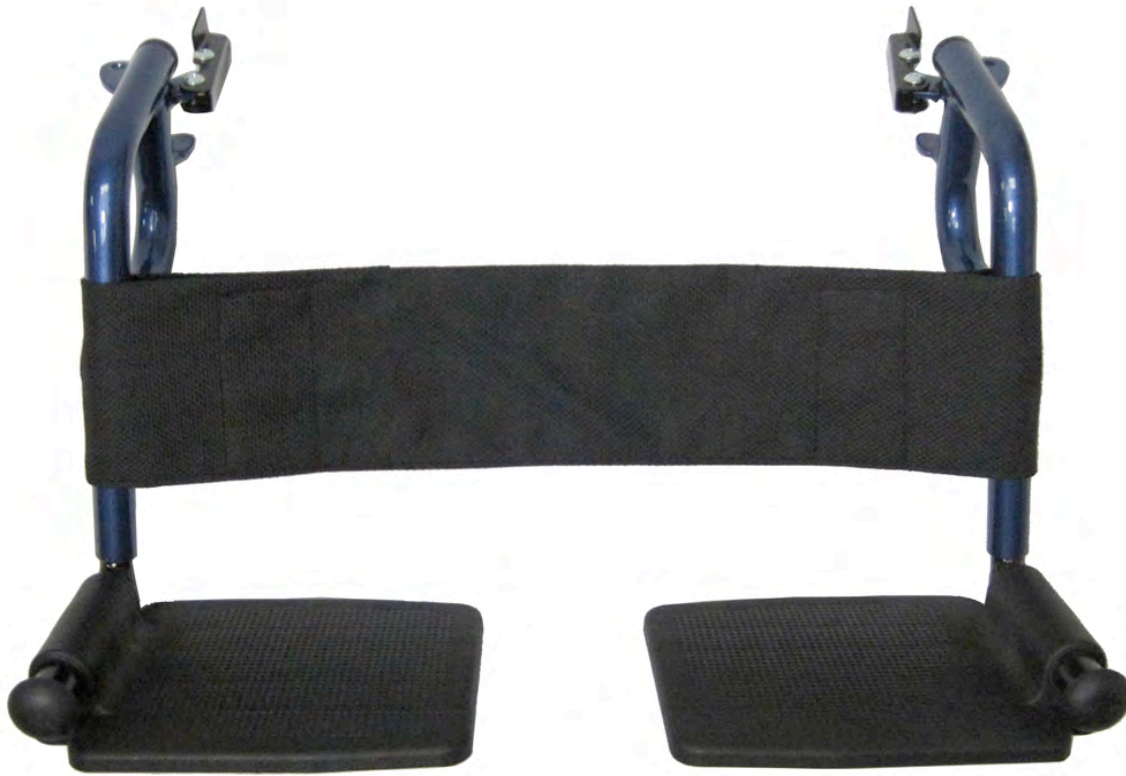

KARMAN HEALTHCARE INC.
19255 San Jose Ave., City of Industry, CA 91748
Tel: 1-800-805-2762 / 626-581-2235 Fax: 626 581-2335
www.karmanhealthcare.com Email: info001@karmanhealthcare.com



Push Handles / Push Bars

Information

HD-XO202 – push handles for stand-up XO-101 / XO-202

PB-115-16 – foldable rigidifying push bars for 16” width

PB-115-18 – foldable rigidifying push bars for 18” width



Push Bars



← PB-115

HD-XO202 →



Hill Climber



Foldable Headrest Backrest Extension



Hand Grips



Push Bars

Part #	Description	
HD-XO202	Push handles for stand-up XO-101 / XO-202	
HD-115RED	Handle bar for S-115 Red w/ handgrip	
HD-115SIV	Handle bar for S-115 Silver	
HD-LT1000HB-BD	Foldback handle bar for LT-1000HB	
HD-LT1000HB-BL	Foldback handle bar for LT-1000HB	
HD-LT2019-BD	Foldback handle bar for LT-2019-BD	
HD-LT2019-BL	Foldback handle bar for LT-2019-BL	
HD-LT2019BD	Foldback handle bar for LT-2019-BD	
PB-115-16	Foldable rigidifying push bar for 16" width	
PB-115-18	Foldable rigidifying push bar for 18" width	

Foldable Headrest

Part #	Description	Models Fit	
HR-FLD-115	Multi-directional headrest, 7/8" frame	S105/106/115/101/202	

Backrest Extension

Part #	Description	
BKR-EXT-S105-16	Backrest attachment standard frame thickness, 16"	
BKR-EXT-S105-18	Backrest attachment standard frame thickness, 18"	
BKR-EXT-S305-16	Backrest attachment standard frame thickness, 16"	
BKR-EXT-S305-18	Backrest attachment standard frame thickness, 18"	

Hill Climber

Part #	Description	
HCB305	Hill-climber attachment for S-305.	

Hand Grips

All handgrips will fit specific frame width, ask if incompatible.

Part #	Description	
HG-7	Hand grip for 7/8" tube. (ebay grooved w/ reflector)	
HG-8	Hand grip 1" for KM-8000 (triangle-texture?)	

KARMAN HEALTHCARE INC.
19255 San Jose Ave., City of Industry, CA 91748
Tel: 1-800-805-2762 / 626-581-2235 Fax: 626 581-2335
www.karmanhealthcare.com Email: info001@karmanhealthcare.com



SB22



- Intended for use on Manual Wheelchairs & Transport Chairs
- Deluxe pinch buckle two-piece seat belt.
- Enable to fit a variety of different manufacturers' chairs.
- 45" long, 2" webbing.
- Black Color.

SB88



- Intended for use on manual wheelchairs & transport chairs
- Deluxe pinch buckle one-piece seat belt
- Enable to fit a variety of different manufacturers' chairs.
- 60" long, 2" webbing
- Black Color

SB99



- Intended for use on Power chairs & Scooters
- Deluxe push button (Auto Style) seat belt.
- Enable to fit a variety of different manufacturers' chairs.
- 60" long, 2" webbing, 1/2 " hole.
- Black Color.



KARMAN HEALTHCARE Inc.

19255 San Jose Avenue . City of Industry . CA 91748-1418
Order. 1-800-805-2762 Tel. 626-581-2235 Fax. 626-581-2335

KARMAN HEALTHCARE INC.
19255 San Jose Ave., City of Industry, CA 91748
Tel: 1-800-805-2762 / 626-581-2235 Fax: 626 581-2335
www.karmanhealthcare.com Email: info001@karmanhealthcare.com



Upholstery – Ergonomics and VIP
Information

UB-S115 – removable upper back for 105 & 115 18” AEGIS Coating

UB-S115N - removable upper back for 105 & 115 16” AEGIS Coating

UB-S115W - removable upper back for 115W 20” AEGIS Coating

UB-S305 - removable upper back for 305 18” AEGIS Coating

UB-S305N - removable upper back for 305N 16” AEGIS Coating

UB-VIP515 - removable upper back for VIP-515 18”

UB-VIP515N - removable upper back for VIP-515 16”

US-S115 - removable upper seat for 105 & 115 18” AEGIS Coating

US-S115N - removable upper seat for 105 & 115 16” AEGIS Coating

US-S115W - removable upper seat for 115W 20” AEGIS Coating

US-S305 - removable upper seat for 305 18” AEGIS Coating

US-S305N - removable upper seat for 305N 16” AEGIS Coating

US-VIP515 - removable upper seat for VIP-515 18” AEGIS Coating

US-VIP515N - removable upper seat for VIP-515 16” AEGIS Coating

Anti-Bacterial Upholstery For Manual Wheelchairs

UB-S115 – removable upper back for 105 & 115 18” AEGIS Coating

UB-S115N - removable upper back for 105 & 115 16” AEGIS Coating

UB-S115W - removable upper back for 115W 20” AEGIS Coating

US-S115 - removable upper seat for 105 & 115 18” AEGIS Coating

US-S115N - removable upper seat for 105 & 115 16” AEGIS Coating

US-S115W - removable upper seat for 115W 20” AEGIS Coating



Upholstery

All upholstery may only fit certain models, ask ahead for measurements

Part #	Description	Notes
UB-2016APT	Upgraded 20" x 16" vinyl back	
UB-2018APT	Upgraded upholstery 20" x 18"	
UB-2218APT	Upgraded 22" x 18" back vinyl airport	
UB-8020F	20" back for 8020F	
UB-S115	Removable upper back S-105/115	
UB-S115N	Removable upper back S-105N/115N	
UB-S305	Removable upper back for S-305	
UB-S305N	Removable upper back for S-305N	
UB-VIP515	Removable upper back for VIP515	
UB-VIP515N	Removable upper back for VIP515N	
UB1618B	Vinyl Back 16"x18" E&J compatible	
UB1618-P	Vinyl back 16" x 18" E&J Blk w/ pocket	
UB1618U	Vinyl back 16" x 18" E&J Blue	
UB1618U-P	Vinyl Back 16" x 18" E&J Blue w/pocket	
UB1817	Back 18" x 17" for KN800	
UB1817LT	Back upholstery 18x17 KM8020	
UB1818B	Vinyl back 18x18 black E&J comp	
UB1818B-P	Vinyl back 18x18 blk E&J w/ pocket	
UB1818U	Vinyl back 18" E&J darkblue w/o pkt	
UB1818U-P	Vinyl back 18" E&J darkblue w/pocket	
UB2018	Back 20x18 for KN920W	
UB2218	Back 22" x 18"	
UB2418	Back 24 x 18 for KN924W	
US-1816APT	Upgraded 18" seat width 4side cover	
US-2016APT	Upgraded 20x16 seat upholstery	
US-2218APT	Upgraded 22x18 vinyl seat airport	
US-8020F	20" for 8020F seat	
US-S115	Removable upper seat S105/115	
US-S115N	Removable upper seat S105/115N	
US-S305	Removable upper seat S-305	
US-S305N	Removable upper seat S-305N	
US-VIP515	Removable upper seat VIP-515	
US-VIP515N	Removable upper seat VIP-515N	
US1616B	Vinyl seat 16x16 black E&J	
US1616LT	Seat 16x16 for T-2700N	
US1616U	Vinyl seat 16x16 blue E&J	
US1816B	Vinyl seat black E&J compatible	
US1816LT	18x16 seat polyester for KM8020	
US1816U	Vinyl seat 18x16 blue E&J	
US1818LT	Seat 18x18 lt wt for KM8020	
US2018	Seat 20x18 for KN920W	


KARMAN HEALTHCARE INC.
19255 San Jose Ave., City of Industry, CA 91748
Tel: 1-800-805-2762 / 626-581-2235 Fax: 626 581-2335
www.karmanhealthcare.com Email: info001@karmanhealthcare.com



Seat Belts

Part #	Description	Notes
SB22	Pinch buckle. 2-piece. 45" x 2"	
SB88	Pinch buckle wrap-around. 1-piece. 60" x 2"	
SB99	Push button Auto-style. 60" x 2" 1/2" hole	

Upholstery

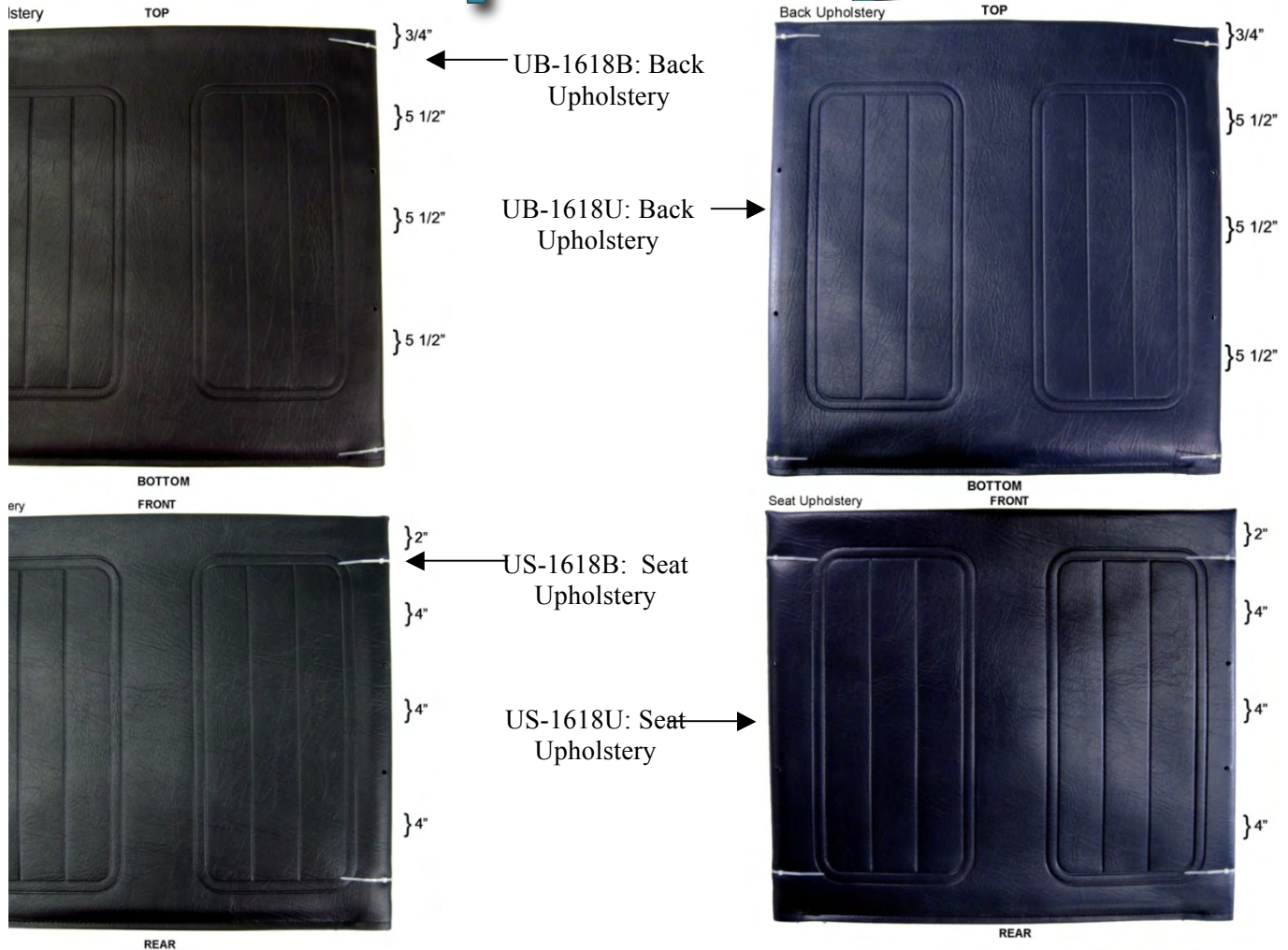

UB-S115 & US-S115:
Looks same for all
Ergonomics

UB-VIP515: Upper
back, lower back &
head cushion



Seat Belts



SB22



AP99



RW-MGN-SY

Magnesium Alloy Wheels

- For everyday/active use!
- Magnesium frame stronger than steel, aluminum or Titanium!
- Magnesium alloys allow wheels to perform in extreme conditions.
- Absorbs vibrations and shocks
- Magnesium is used in car engines as a stronger, much more lightweight substitute.
- Quick-Release axle
- 1" x 1" Pneumatic Tires, 24" Wheel
- The pair of wheels & tires weigh only 7.5 lbs



KARMAN HEALTHCARE Inc.

19255 San Jose Avenue, City of Industry. CA 91748-1418

KARMAN HEALTHCARE INC.
19255 San Jose Ave., City of Industry, CA 91748
Tel: 1-800-805-2762 / 626-581-2235 Fax: 626 581-2335
www.karmanhealthcare.com Email: info001@karmanhealthcare.com



Order. 1-800-805-2762 | Tel. 626-581-2235 | Fax. 626-581-2335

RW-MGN-SY: Magnesium (Smooth Chrome Rims)



Specifications

- A. 24" x 1" Pneumatic Inflatable Tire
- B. 1" Thick
- C. 1.5" Total Depth
- D. 3 lbs. each, tire & rim

- For everyday/active use!
- Magnesium rim comes attached to inflatable 3-spoke tire.
- Hand grip made from ultra-light weight magnesium alloys to reduce weight and friction on hand.
- Smooth chrome finish drastically reduces friction.
- Tires feature bright yellow trim for greater exposure.
- Solid handrim with no gaps helps ensure constant smooth surface.
- Lightweight 3-spoke design to reduce overall weight.
- Quick-Release axle for easy transporting.
- 1"x 1" Pneumatic Tires, 24" Wheel
- The pair of wheels & tires weigh only 6 lbs



KARMAN HEALTHCARE INC.
 19255 San Jose Ave., City of Industry, CA 91748
 Tel: 1-800-805-2762 / 626-581-2235 Fax: 626-581-2335
www.karmanhealthcare.com Email: info001@karmanhealthcare.com



Rear Wheels



← RW-MGN-SY



RW-VLCN-SG →



← RF-115



Low-Profile Wheel →



← High-Profile Wheel

Tires



Low-Profile
Tire



High-Profile
Tire

Large Manual Rear Wheels

Please call ahead if wheels are compatible with your wheelchair.

Part #	Description	Notes
R241	Rear mag wheels black	
RC-S115	Reflection Wheel cover for S-ergo	
RF-115	Reflection rear wheel cover pair	
RW-802022	Rear wheel for KM8020F20/22"	
RW-9020L	20 x 1 rear spoke wheel for KM9020L	
RW-950	Rear wheel 22 x 1 for LT-950	
RW-MGN-SG	Magnesium wheel 24 x 1 silver/grey	
RW-MGN-SY	24 x 1 magnesium q/r yellow tire	
RW-S105	Rear spoke wheel 24" PU tire, S-115	
RW-S305Q	Rear spoke wheel, 24" PU tire S-305	
RW-VLCN-SG	24 x 1 vulcan q/r grey tire silver rim	
RW12	Rear Mag Wheels 12" PU tire	
RW-221	Rear mag wheels 22 x 1 w/bearing	
RW241	Rear mag wheel, 24 x 1	
RW88	Rear mag wheel, 24 x 1 3/8 fpr S-305	
T241	24" x 1" Rubber Tire	
T24134	24" x 1 3/4" PU Tire	
T24183	24" x 1 3/8" PU tire	
TC-101	Tire Changer for Snap-on tires	
TIRE-20	20" x 1 3/8" PU tire	
TIRE-22	22" x 1 3/8" PU tire	
TIRE-PU-24	24" x 1" PU tire for KM8020	
TOILET-MINI	Mini-toilet disposable	
TP24	24" PU tire	
TU24183	23" x 1 3/8" inner tube	
TUBE-6	6" x 1 1/4" Inner Tube	
URETHANE-24	24 x 1 3/8 urethane tires	
WHEEL22SPOKE	22x1 3/8 spoked wheels w/ urethane	
WHEEL24MAG	24x1 3/8 magwheel PU tire	
WHEEL24SPOK-QR	24" rear spoke whl qk rl urethane tire	

Inner Tube

Please call ahead if tubes are compatible with your wheel.

Part #	Description	Tread Type	Notes
IT58	10x3 (260x85) (300-4) inner tube		
IT74	280x250-4 & 9x3.5-4 inner tube		

KARMAN HEALTHCARE INC.
 19255 San Jose Ave., City of Industry, CA 91748
 Tel: 1-800-805-2762 / 626-581-2235 Fax: 626 581-2335
www.karmanhealthcare.com Email: info001@karmanhealthcare.com



KARMAN HEALTHCARE INC.
 19255 San Jose Ave., City of Industry, CA 91748
 Tel: 1-800-805-2762 / 626-581-2235 Fax: 626 581-2335
www.karmanhealthcare.com Email: info001@karmanhealthcare.com

RF-115: Reflector Wheel Cover



- Doubles as a bright silver safety reflector for traffic at nighttime as well as a guard for users hands from the actual spokes of the wheels.
- Gap at center allows for quick-release function to still be accessible.
- 3 separate 1" silver reflectors highlight a 3" bright silver circular surface that also incorporates an encouraging quote ("Discover the path and make it yours")
- Made of high strength composite materials for durability and to keep chair lightweight.
- For S-Ergo line of wheelchairs only.
- One-Size fits all!

Specifications

- E. Meant for 24" x 1" Tire
- F. 0.3 lbs. each
- G. 1" Thick Outer Ring Bright Strip
- H. 3" Thick Inner Ring Bright Strip
- I. Only for S-Ergo Series Wheelchairs





RF-115

Reflective

Cover/Spoke Guard

- Doubles as a bright silver safety reflector for traffic at nighttime as well as a guard for users hands from the actual spokes of the wheels.
- Gap at center allows for quick-release function to still be accessible.
- 3 separate 1" silver reflectors highlight a 3" bright silver circular surface that also incorporates an encouraging quote ("Discover the path and make it yours")
- Made of high strength composite materials for durability and to keep chair lightweight.
- For S-Ergo line of wheelchairs only.
- One-Size fits all!



KARMAN HEALTHCARE Inc.

19255 San Jose Avenue. City of Industry, CA 91748-1418
Order.1-800-805-2762 Tel.626-581-2235 Fax.626-581-2335

KARMAN HEALTHCARE INC.
19255 San Jose Ave., City of Industry, CA 91748
Tel: 1-800-805-2762 / 626-581-2235 Fax: 626 581-2335
www.karmanhealthcare.com Email: info001@karmanhealthcare.com



TIRE-24/TUBE-24 – PNEUMATIC TIRES AND TUBES



KARMAN HEALTHCARE Inc.

19255 San Jose Avenue . City of Industry . CA 91748-1418
Order.1-800-805-2762 Tel.626-581-2235 Fax.626-581-2335

KARMAN HEALTHCARE INC.
19255 San Jose Ave., City of Industry, CA 91748
Tel: 1-800-805-2762 / 626-581-2235 Fax: 626 581-2335
www.karmanhealthcare.com Email: info001@karmanhealthcare.com



KARMAN HEALTHCARE INC.
19255 San Jose Ave., City of Industry, CA 91748
Tel: 1-800-805-2762 / 626-581-2235 Fax: 626 581-2335
www.karmanhealthcare.com Email: info001@karmanhealthcare.com

RW-VLCN-SG: Vulcan (Ergonomic Grip Wheels)



Specifications

- J. 24" x 1" Pneumatic Inflatable Tire
- K. 1" Thick
- L. 1.5" Depth
- M. 4 lbs. each, tire & rim



- For everyday/active use!
- Vulcan rim comes attached to inflatable spoke tire.
- Vulcan grip made from aluminum alloys to reduce weight and friction on hand.
- With wider grip, users will be able to stay much cleaner without gloves.
- Rims have greater surface area allowing for
- Lighter ergonomic grip will reduce stress on the user's wrists and other joints
- Improved propulsion, less effort per push, gets you further on less energy.
- Solid handrim with no gaps helps ensure constant smooth surface.
- Lightweight spoke design to reduce overall weight.
- Quick-Release axle for easy transporting.
- 1"x 1" Pneumatic Tires, 24" Wheel
- Push rims comparable with Natural-Fit rims but doesn't burn the hands!
- The pair of wheels & tires weigh only 8.0 lbs





RW-VLCN-SG

Vulcan (Ergonomic Grip Wheels)

- For everyday/active use!
- Vulcan grip made from aluminum alloys to reduce weight and friction on hand.
- With wider grip, users will be able to stay much cleaner.
- Lighter grip will reduce stress on the users wrists and other joints
- Lightweight spoke design to reduce overall weight
- Quick-Release axle for easy transporting
- 1"x 1" Pneumatic Tires, 24" Wheel
- Push rims comparable with Natural-Fit rims but doesn't burn the hands!
- The pair of wheels & tires weigh only 8.0 lbs

