



KN-TV10A

6061 T6 Aircraft-grade Aluminum
Super Light Weight Travel Chair

User Guide and Safety Tips



KARMAN HEALTHCARE INC.
19255 San Jose Ave.
City of Industry, CA 91748

KARMAN HEALTHCARE INC.
www.karmanhealthcare.com

INTRODUCTION

Congratulations on the purchase of your KARMA product. This well-designed wheelchair provides excellent customer satisfaction at a very competitive price. Before using your wheelchair, please read the following material, particularly the Safety Tips. It is important that you consult your healthcare professional and authorized Karma supplier to make certain that your chair is properly adjusted for your safety.

WARNING: The user should be evaluated by a healthcare professional so that the user's physical condition and limitations are considered to assure safe operations of the wheelchair in the performance of daily activities.

A complete inspection, safety check and servicing should be made by an authorized Karman supplier at least once a year.

Warning

This product may expose you to chemicals, which is known to the State of California to cause cancer and birth defects or other reproductive harm. For more information, visit www.P65Warnings.ca.gov

Design and specifications are subject to change without notice.

WARRANTY: KARMAN WHEELCHAIR LIMITED WARRANTY

This warranty is extended only to the consumer, and does not cover commercial use (such as rental or institutional use) and is not transferable. Karma warrants main frame on the KN-TV10A series travel chairs against defects in materials and workmanship for one year of the original purchaser, warrants parts and components (including frame components, caster plates, armrests, backrests, axle plates, etc.) for one year from purchase date. All tires and tubes, upholstery, and armrest pads/foams are warranted for three months. Parts or materials which are subject to normal wear and which must be replaced/repairs are the owner's responsibility and are not covered by warranty. It is recommended that wheelchair be returned to the authorized dealer through whom it was original purchased.

If within such warranty period, any such product is proven to Karman's satisfaction to be defective, such product shall be repaired or replaced at Karman option. In no case are goods to be returned without first obtaining an R.M.A. (Returned Merchandise Authorization) number issued by Karman and that the part in question is returned insured with freight prepaid to Karman, within the period of warranty. Labor charge incurred in the installation or repair of the above-mentioned parts plus all freight charges are not included in the warranty.

If Karman does not have a completed Warranty Registration Card on file for the consumer's product, then a copy of the invoice with date of purchase must be provided. All Karma wheelchairs are identified by a Q.C.# (Serial #) at time of manufacture. The warranty is void on wheelchairs that have had the serial # tag removed or altered.

Products which have been subject to negligence, abuse, improper storage or handling, improper operation, unauthorized modifications or misuse are not covered by this warranty. Karman will not warrant the frame or any parts damaged when the maximum weight limit is exceeded. Warranty applies in the U.S. only.

Karman Healthcare's sole liability extends to repair or replacement of parts or components as previously specified. The foregoing is in lieu of all warranties expressed, implied or statutory, whether written or oral.

Karman User Instruction Manual 05/2021

Model #: _____ Serial #: _____
Date of Purchase: ____/____/____

WARNING: When reassembling the wheelchair after removing the rear wheels, ensure that the rear axles are locked in place to prevent wheels from falling off.

Street Use

Your travel chair should not be operated on roads and streets except when using crosswalks. At such times, use only marked pedestrian crosswalks and proceed with extreme caution.

Transferring

Never stand on the footrest footplates while transferring because the chair may tip over. Always engage the wheel locks before transferring in or out of your wheelchair.

Wheel Locks

Wheel locks are designed to prevent the rear wheels from turning when the chair is stopped. It is not intended to slow down the chair when it is moving. Adjust the wheel locks after making any changes to the rear axle sleeves position.

Weight Limit

Maximum user weight is 190 lbs.

WARNING: This wheelchair has not been approved as a seat in a motor vehicle. Always transfer to an approved vehicle seat.

WARNING: Do not tie down or attach anything to the wheels in any way. This could cause tipping and possibly result in injury or damage to the chair.

WARNING: Attempting inclines steeper than 6 degrees can cause your chair to tip over. Use of anti-tipping device is highly recommended for inclines.

WARNING: Do not operate your chair on hilly or rough terrain or in conditions beyond your ability. They could cause you to lose control of your chair and should be avoided.

WARNING: Engage wheel locks when using this wheelchair in an elevator or wheelchair lift, and when transferring.

WARNING: Before transferring to or from travel chair, place chair on a stable, level surface and engage wheel locks.

SPECIFICATIONS



- | | |
|--------------------------------------|------------------------------------|
| 1. Push Handle Backrest Tube | 8. Rear Wheel Lock Brake |
| 2. Padded Folding Armrests | 9. Rear Wheel Lock Brake Handle |
| 3. Backrest | 10. Folding Chain (Push/Pull Down) |
| 4. Padded Seat | |
| 5. Flip-up Footrest | |
| 6. Front Caster w/ Polyurethane Tire | |
| 7. Rear Caster w/ Polyurethane Tire | |

OPERATING

1. Opening Travel chair

Lay KN-TV10A travel chair on flat surface toward you on the side (As figure.1) Make sure both the front and rear casters DO NOT align (see figure.2) Lock rear wheel lock brake. Hold Point A (at the edge of the padded seat) then Pull at Point B (push-handle backrest tube) until fully extend in the upright position. Seat the user down on KN-TV10A, then fold down armrest and footrest. Unlock rear wheel lock brake.

WARNING: When unfolding make sure the front casters and the rear casters DO NOT align (see figure.2) or damage to folding mechanism will result if extreme force is applied.



Figure. 1 CORRECT



Figure. 2 WARNING: WRONG

2. Folding Travel chair

Lock rear wheel lock brake. Fold up armrests and footrests, then unseat user from KN-TV10A before folding the chair. Hold Point B (push-handle backrest tube) with one hand, then while push/pull down folding chain with the other hand - - fold the backrest until flat.

GENERAL MAINTENANCE RECOMMENDATIONS

For maximum service, it is recommended that the wheel bearings be cleaned, inspected and regreased about once a year by dealer. Adjust brakes when necessary. Check periodically for loose screws, bolts or spokes and repair immediately.

The best way to preserve the finish is to keep it clean by sponging with a damp cloth and drying immediately with a soft dry cloth. Most of the hinged or movable parts may be lubricated with “White Graphite”.

SAFETY TIPS

WARNING: Never stand on the footrests, for the chair may tip forward.

Opening and Folding the wheelchair

Do not place your fingers between the folding mechanism when opening or folding the wheelchair.

WARNING: TV-10A is to be used by a user with an attendant. Never use this travel chair without an attendant.

FEATURES

1. Aircraft-grade ultra-lightweight aluminum frame: only 15 lbs.
2. Quick folding or unfolding in under 5 seconds
3. Compact and easy to carry over and store.
4. Ideal for shopping, road trips, and air travel.
5. An assistant is required to push the chair from behind.
6. Optional security belt is available.

